

# SoloFit<sup>®</sup> Self-Donning Process




Attach Non-sterile clips to IV pole and position Sterile Tabs



Using single person aseptic technique, don gown and gloves.



Grasp and separate connected Tabs from SoloFit front

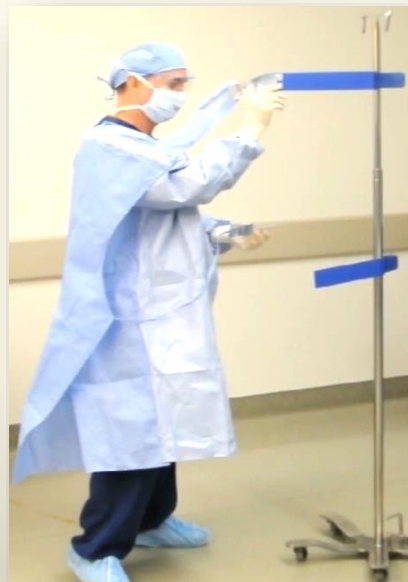

Attach Face and Waist Tabs to corresponding IV pole Tabs



With tabs secured, rotate into the gown, while adjusting gown to shoulders



Adjust shoulder strap and waist ties for customized, comfort fit.

For more information  
Visit: [www.orinnovations.com](http://www.orinnovations.com)

Request samples or place order:  
+1 (619) 210-0725  
+1 (619) 393-0181  
[ori.support@orinnovations.com](mailto:ori.support@orinnovations.com)