

Direct Anterior Hip Drape Guide



1
Without removing drape Tabs, unfold drape
Remove incision window backing from
surgical side.



2
Unfold drape and place incision window over surgery
site. Press firmly to secure incision window.
Release tabs.



3
Unfold DAHIP drape over lower extremities



4
Unfold DAHIP drape over upper torso
ensuring total coverage



5
One-piece large drape size, standardizes draping
process, while significantly reducing excess draping
motions, prep times, and disturbance to airflow.

Direct Anterior Hip Drape (P/N: ORI-SDL-DAHIP)

Unique one-piece drape with bilateral fenestration ports and collection pouches, improve OR workflow and patient outcomes with less risk of infection.

Request product information and ordering:
Main: +1 (619) 210-0725 | Fax: +1 (619) 393-0181
Email: ori.support@orinnovations.com

Operating Room Innovations
5694 Mission Center Rd Ste 602
San Diego, CA 92108
www.orinnovations.com