



Open Drape so incision windows are above the leggings. Break legging tabs.



Slide leggings over patient's feet. Advance drape towards pelvis.



Remove incision window backing (this step can be completed in Step 1 if preferred) Place incision window over surgical site.



Unfold remaining upper and lower drape. Position limb control device into port sleeve.



Advance limb control device through drape and port sleeve.

Lower Extremity Adaptable Drape
(P/N: ORI-SDL-LEAD)
Unique one-piece drape with four integrated sleeve ports, designed to work with limb positioning device.

Request product information and ordering:
Main: +1 (619) 210-0725
Fax: +1 (619) 393-0181
Email: ori.support@orinnovations.com
Operating Room Innovations
5694 Mission Center Rd, Ste 602
San Diego, CA 92108
www.orinnovations.com