

Universal Upper Extremity Drape (Beach Chair) Draping Guide

- ① Remove the protective paper layer to reveal the drape's adhesive surface. Adhere to the directional indicators marked 'Ceiling' or 'Floor' to ensure correct orientation and placement of the drape.



- ④ Detach the tab from the port sleeve to facilitate the advancement of the limb positioner. Ensure careful handling to maintain the integrity of the sleeve and positioner.




- ② Advance the drape's fenestration over the limb of the operative site. Methodically unfold the superior and lateral aspects of the drape to establish a sterile field over the designated surgical area.



- ⑤ Extend the limb positioner through the port sleeve, ensuring it is fully deployed. Carefully unfold the drape to achieve comprehensive sterile coverage of the surgical area.



- ③ Securely affix the drape around the shoulder, ensuring a snug and stable fit. Position the fluid control pouches strategically to efficiently manage and contain surgical fluids.

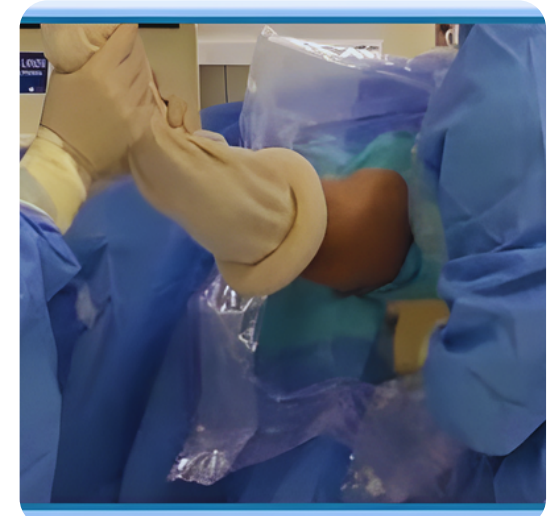

Upper Extremity Drape (P/N: ORI-SDL-UPXD)

This unique one-piece upper extremity shoulder drape, measuring (162"x102") inches, is equipped with an integrated fluid collection pouch and port sleeves for a limb control device. It is specifically designed to significantly reduce draping motions and preparation times while minimizing disturbance to the surgical airflow environment.

For more information or to place an order:
Main: +1 (619) 210-0725 | Fax: +1 (619) 393-0181
Email: contact@orinnovations.com

Operating Room Innovations
5694 Mission Center Rd Ste 602
San Diego, CA 92108
www.orinnovations.com